

4" x 5.5"

MimyX™
cream

For Topical Dermatological Use Only

Rx Only

DESCRIPTION: MimyX™ Cream is a fragrance free, preservative free, water-based emulsion formulated for the management and relief of the burning and itching experienced with various types of dermatoses, including atopic dermatitis, allergic contact dermatitis and radiation dermatitis. MimyX™ Cream helps to relieve dry, waxy skin by maintaining a moist wound and skin environment, which is beneficial to the healing process.

INDICATIONS FOR USE: MimyX™ Cream is indicated to manage and relieve the burning and itching experienced with various types of dermatoses, including atopic dermatitis, allergic contact dermatitis and radiation dermatitis. MimyX™ Cream helps to relieve dry, waxy skin by maintaining a moist wound and skin environment, which is beneficial to the healing process.

CONTRAINDICATIONS: MimyX™ Cream is contraindicated in persons with a known hypersensitivity to glycerin.

WARNINGS: MimyX™ Cream should not be used prior to extended exposure to the sun. In radiation therapy, MimyX™ Cream may be applied as indicated by the treating Radiation Oncologist. Do not apply within four hours prior to a radiation session.

PRECAUTIONS AND OBSERVATIONS:

- MimyX™ Cream is for external use only.
- MimyX™ Cream does not contain a sunscreen and should not be used prior to extended exposure of the sun.
- If clinical signs of infection are present, appropriate treatment should be initiated;

- use of MimyX™ Cream may be continued during the anti-infective therapy.
- If the condition does not improve within 10-14 days, consult a physician.
- **Keep this and other similar products out of the reach of children.**
- MimyX™ Cream may dissolve fuchsin when this dye is used to define the margins of the radiation fields to be treated.

ADVERSE REACTIONS: Call your doctor for medical advice about side effects. You may report side effects to the FDA at 1-800-FDA-1088.

INSTRUCTIONS FOR USE: Apply MimyX™ Cream to the affected areas three times per day (or as needed or directed by a physician), and massage gently into the skin. If the skin is broken, cover MimyX™ Cream with a dressing of choice.

INGREDIENTS: Purified water, hydroxyethyl cellulose, sodium carbomer, carbomer, xanthan gum, glycerin, pentylene glycol, palmitamide MEA, acetamide MEA, betaine, sarcosine, olive oil, palm glyceride, vegetable oil, hydrogenated lecithin, squalane.

HOW SUPPLIED: MimyX™ Cream is available in a 100g tube.

Discard within 180 days of opening or at the expiration date.

Store at 15°C to 30°C (59°F to 86°F). Do not freeze.

Manufactured for:

Artesa Labs
13785 Research Blvd., Suite 125
Austin, TX 78750
827528 v1 Rev 02/18

**Rx Only - Prescription Medical Device:
Federal Law restricts this device to sale
by or on the order of a physician.**